2023

# Workshop Defiance

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OA scharnier  
Unicompartmentale artrose

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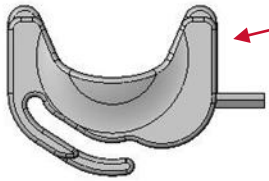
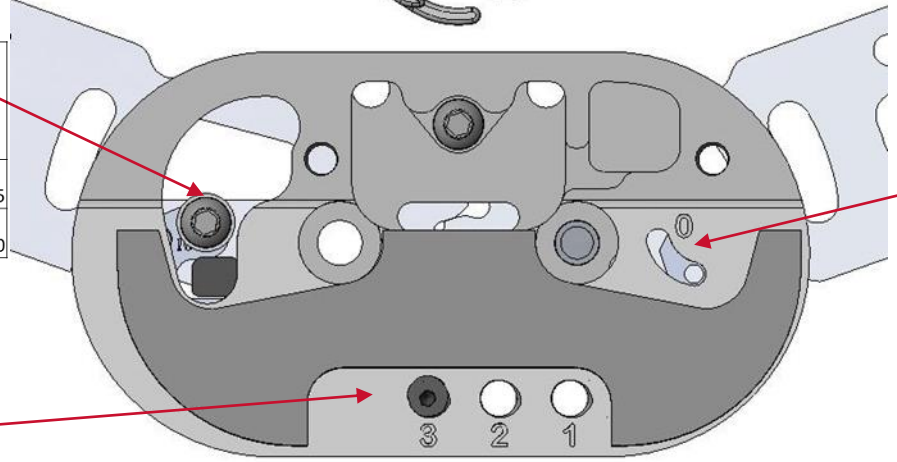
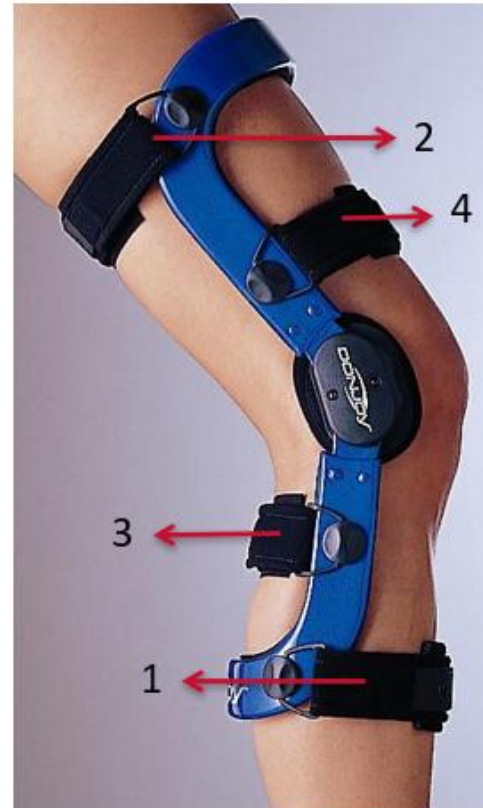
- **Steel Plate (FourcePoint Plus)**
  - Football only
  - \$20 up charge
- **4/5 Power Cap**

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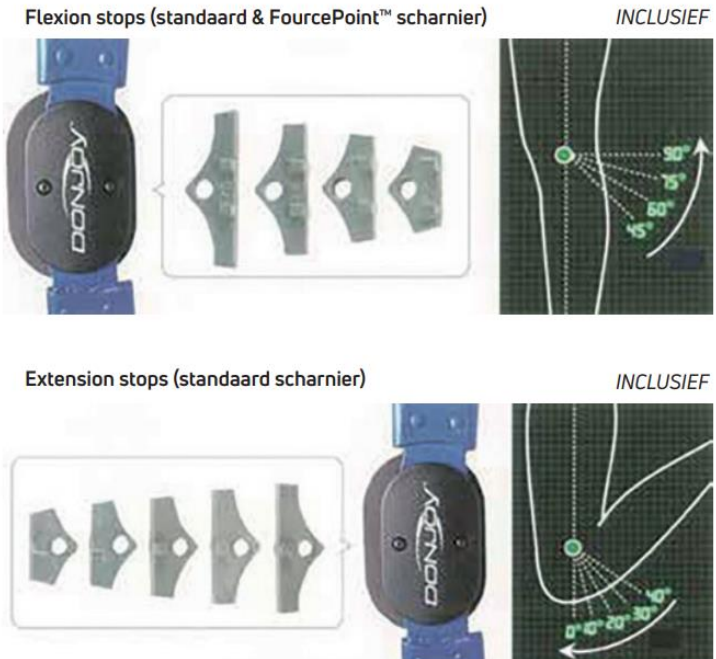
❖ Gel condyl

❖ Atrophy padding Kit

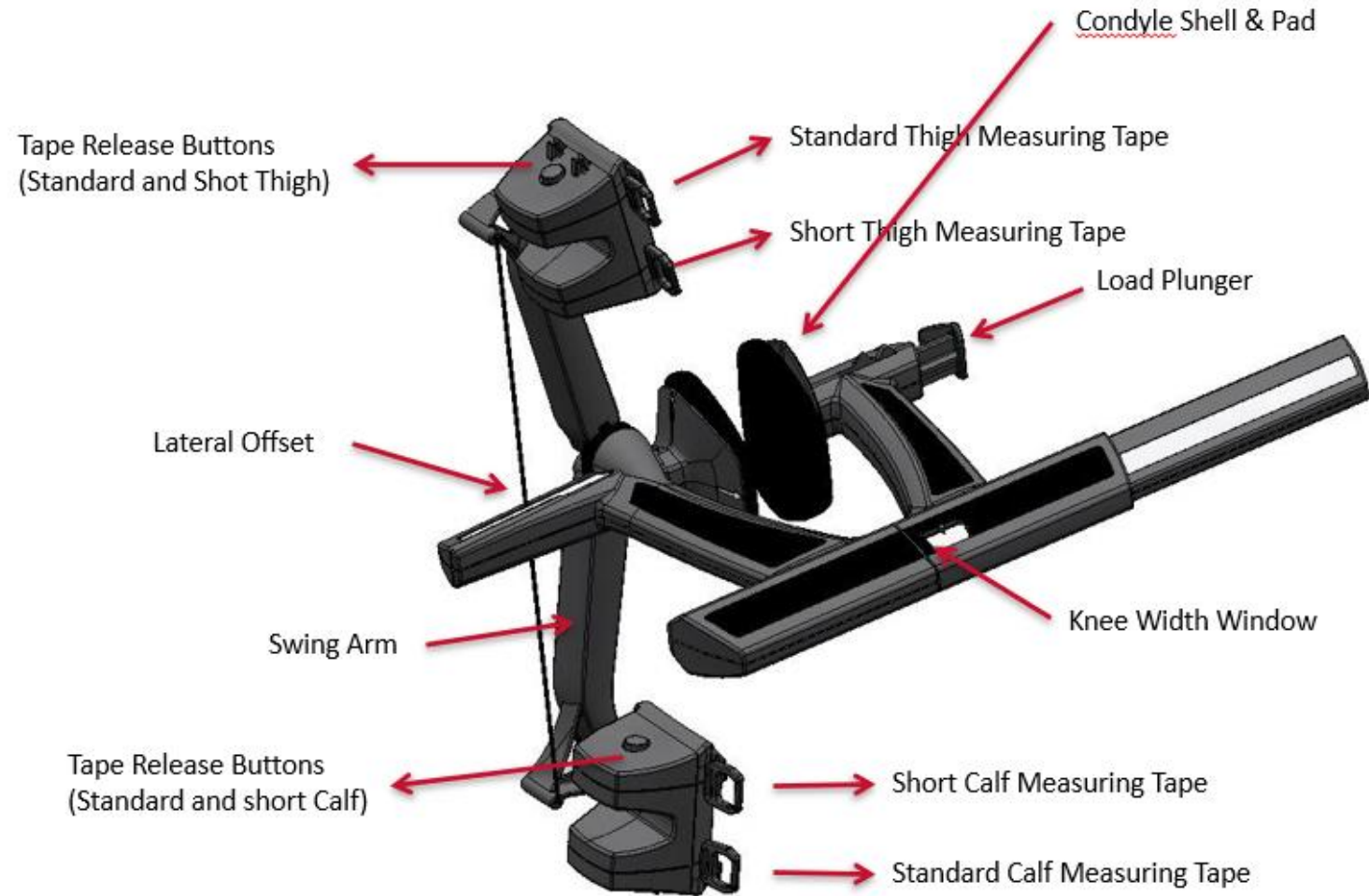
### ❖ TRU-PULL Advanced Attachment (TPAA) – Patella instabiliteit

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Address: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

### SHIPPING INFORMATION: Same as "BILL TO"

Name: \_\_\_\_\_

Address: \_\_\_\_\_

City: \_\_\_\_\_ State: \_\_\_\_\_ Zip: \_\_\_\_\_

### PRODUCT SELECTION: CCMI Mark III

Frame:  DEFIANCE® III  DEFIANCE®

#### Hinge:

FourcePoint™ (ACL only)

Low Profile / Standard (select strapping sequence below)

Adjustable OA (select strapping sequence below)

#### Strapping Sequence:

ACL  PCL  CI

OA Affected Compartment:  Medial  Lateral

Degrees of Correction: \_\_\_\_\_ (1o to 7o) (Standard= 3o)

Frame kiezen

Scharnier kiezen

Kiezen strapping

Aangedane  
compartiment OA en  
eventuele ingestelde  
graden

### FOR DJO USE ONLY:

Order #: \_\_\_\_\_

Brace #: \_\_\_\_\_

### PATIENT INFORMATION:

Name: \_\_\_\_\_

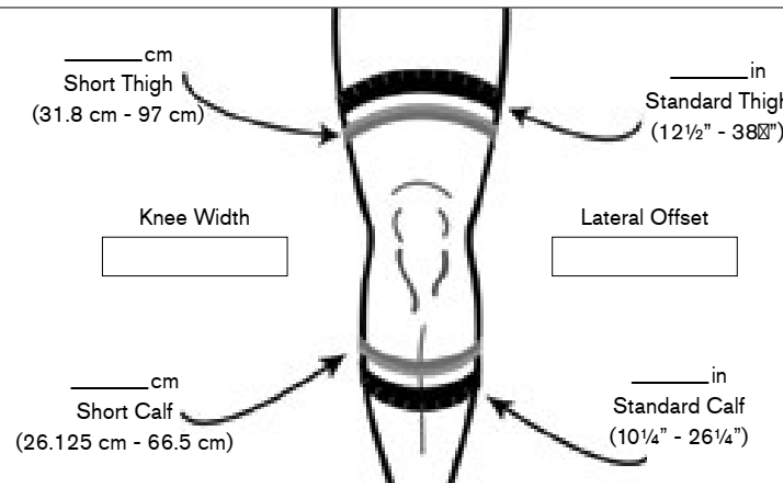
Age: \_\_\_\_\_ Height: \_\_\_\_\_ Weight: \_\_\_\_\_

Knee Measurement:  Right  Left  Reverse

Measured By: \_\_\_\_\_

New Brace  Remeasurement/Repair  Refurbish

Info patiënt

Maatname via  
CCMIII

Note: Check both short and standard measurements for ski boot option. Best to measure with Ski Boot On.

Kleur kiezen

**COLORS:**

- Black Gloss    Seafoam Gloss    True Red Gloss    Bright Blue Gloss
- Black Matte    Emerald Gloss    Fuchsia Gloss    Cotton Candy Gloss
- Smoke Matte    Volt Gloss    Crimson Gloss    Olive Drab Green Matte
- White Gloss    Bumble Gloss    Grape Gloss    Flat Dark Earth Matte
- Navy Gloss    Safety Gloss    Velvet Gloss    Desert Tan Matte
- Powder Gloss

**METALLIC GLOSS:**

- Metallic Black    Metallic White Gloss    Metallic Forrest Gloss
- Metallic Gunmetal Gloss    Metallic Bright Blue Gloss    Metallic Gold Gloss
- Metallic Silver Gloss    Metallic Navy Gloss    Metallic Tru Red Gloss

**MULTIPLE COLORS:**

Thigh Frame: \_\_\_\_\_

Calf Frame: \_\_\_\_\_

Fade (non-metallic colors only):    Yes    No

**GRAPHICS (extra charge):**

- Stars & Stripes (non-metallic)    Stars & Stripes (metallic)

NOTE: Check both short and standard measurements for ski boot option. Best to measure with ski boot on.

**FEATURES (\*standard option on DEFIANCE® III):**

- Suede Liners/Pads
- Chamois Liners/Pads
- Swooping Thigh (not available on short thigh)\*
- Swiveling Strap Tabs\*
- Metal Caps - PowerCaps (recommended for collision sports)
- Reinforced Carbon Composite Frame
- Legend Strap Ends
- Installed Extension Stop: 0° 10° 20° 30° 40° (circle one)
- Installed Flexion Stop: 45° 60° 75° 90° (circle one)
- Neoprene Suspension Strap Kit

Gratis  
accessoires

**ACCESSORIES (extra charge):**

- High Strength Stainless Steel Gears
- Full Extension Lockout Stop
- Lycra® Suspension Sleeve
- Neoprene Suspension Sleeve
- Sports Brace Cover
- Sili-Grip Strap Pad Kit
- Condyle Pad Kit
- Air Condyle Kit
- Surround Condyle Pads
- Armor Impact Guard
- Pinch Guard (Calf)
- Atrophy Pad Kit
- Tru-Pull® Advanced

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
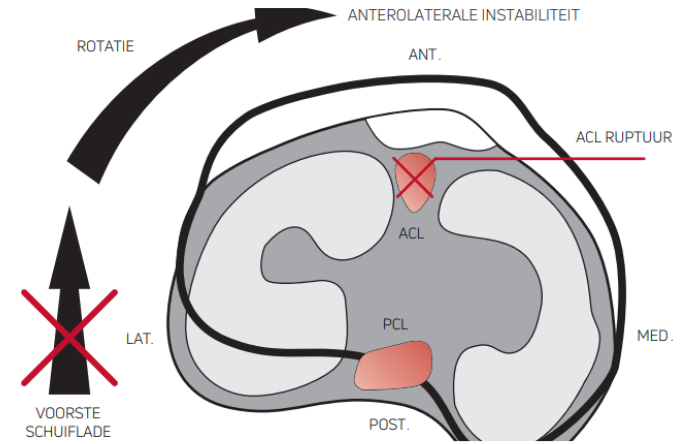
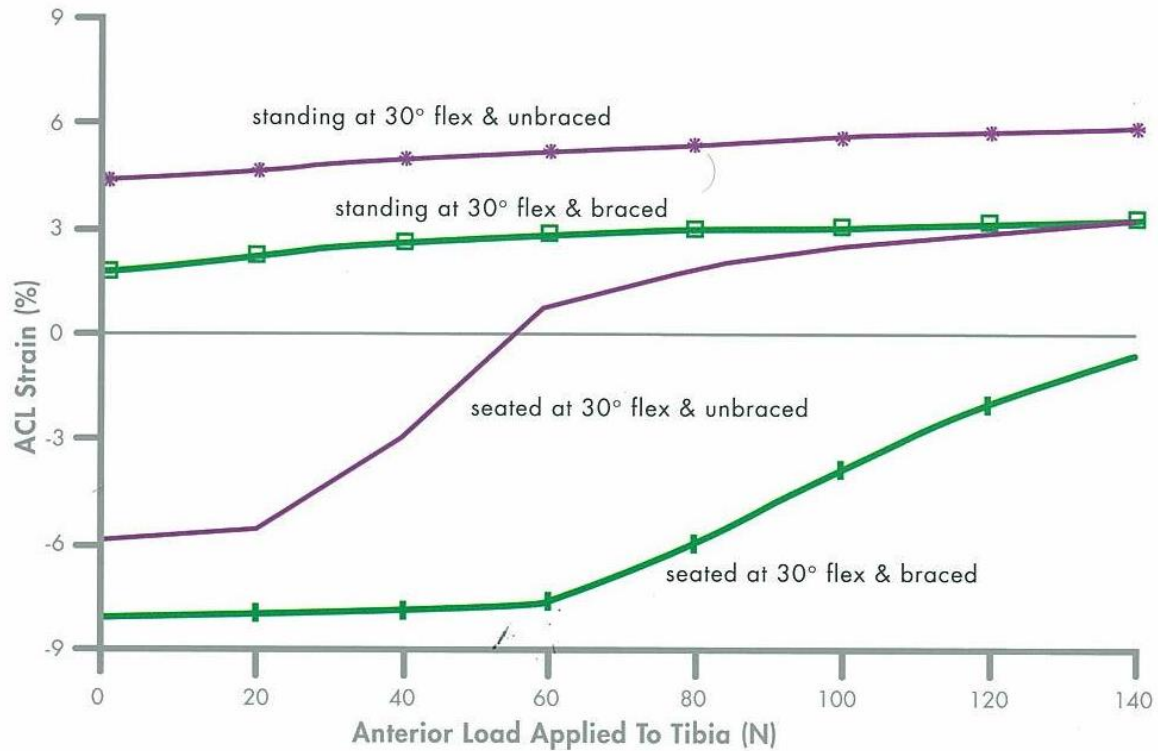
Figure 2: ACL strain values produced by anterior tibial loading (Beynon *et al.* 1995B).

## CONCLUSION

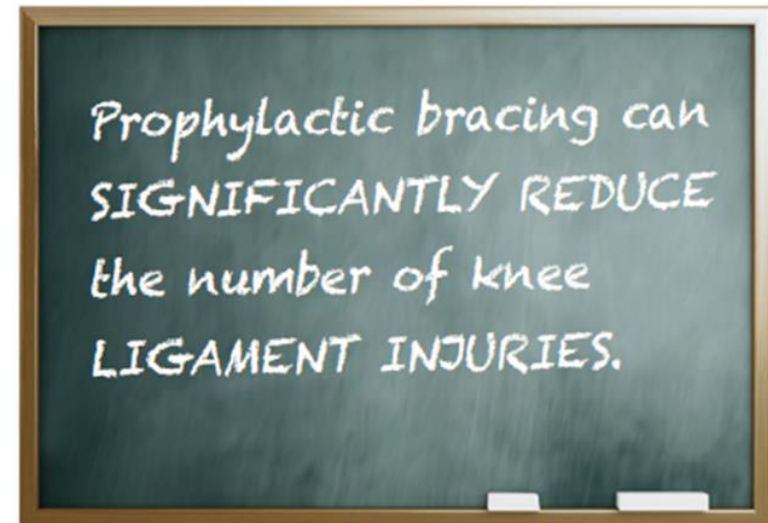Braces utilizing Four-Points-of-Leverage technology effectively reduce ACL strain which could be particularly important during rehabilitation while the graft is remodeling.

## STUDY FOCUS CLINICAL REVIEW

Issue 13 - September 2022  
Bracing & Supports

## KNEE OA PATIENTS REPORT IMPROVED PAIN, FUNCTION AND MOBILITY WITH OA DEFIANCE

### Effects of a semi-rigid knee brace on mobility and pain in people with knee osteoarthritis.

**Authors** Dries T, van der Windt JW, Akkerman W, Kluijtmans M, Janssen RPA.

**Published** J Rehabil Med Clin Commun. 2022 Jul 5;5:2483.

**Date** Jul 2022

**Place of origin** Livit Orthopedie, The Netherlands; Eindhoven University of Technology and Fontys University of Applied Sciences, Eindhoven, The Netherlands.

**Background** Several systematic reviews have quantified the effectiveness of unloader braces in improving clinical outcomes and mechanical leverage. However, to the best of the authors' knowledge, there is a significant research gap in perception-based studies in understanding the short- and long-term consequences of these braces on the tissues in the knee joint, including the cartilage and ligaments.

**Objective** The aim of this perception-based study is to evaluate the short- and long-term effects of conservative therapeutic use of a semi-rigid knee brace for management of patients with knee osteoarthritis, using patient-reported outcomes (PROM).

**Tested products** Donjoy OA Defiance  
Ottobock Agillium Reactive

### Study design & methods

#### Perception-based evaluation study.

**Subjects:** 381 patients with Knee OA who were allocated treatment with a semi-rigid knee brace (both new users and repeat users).

The diagnosis of OA was confirmed by a medical doctor (MD) for all patients. The diagnosis was mostly made based on X-rays and/or MRI.

**Methods:** Patients were provided with either a Donjoy OA Defiance brace or a Ottobock Agillium Reactive brace (both selected for their unloading capabilities).

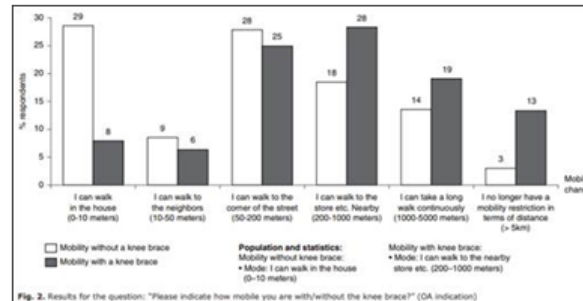
Patients were asked to complete a questionnaire about the effectiveness of the brace after wearing it for 3 weeks.

#### Outcomes:

- Mobility, assessed using an ordinal scale with and without use of the knee brace,
- Pain symptoms,
- Overall daily functioning.

### Results

- The results show considerably improved mobility while using a knee brace in different mobility groups.
  - The number of respondents who were limited to their home environment decreased from 29% without using a knee brace to 8% while using a knee brace.
  - The number of respondents who were able to walk to a nearby shop increased from 18% without using a knee brace to 28% while using a knee brace.
  - The group experiencing no mobility restrictions increased from 3% without using a knee brace to 13% while using a knee brace.

- 54% of respondents reported a reduction in pain symptoms.
- 62% of respondents reported an improvement in overall daily functioning while using a knee brace.
  - 29% indicated that daily functioning improved significantly, and 7% significantly.


### Conclusion

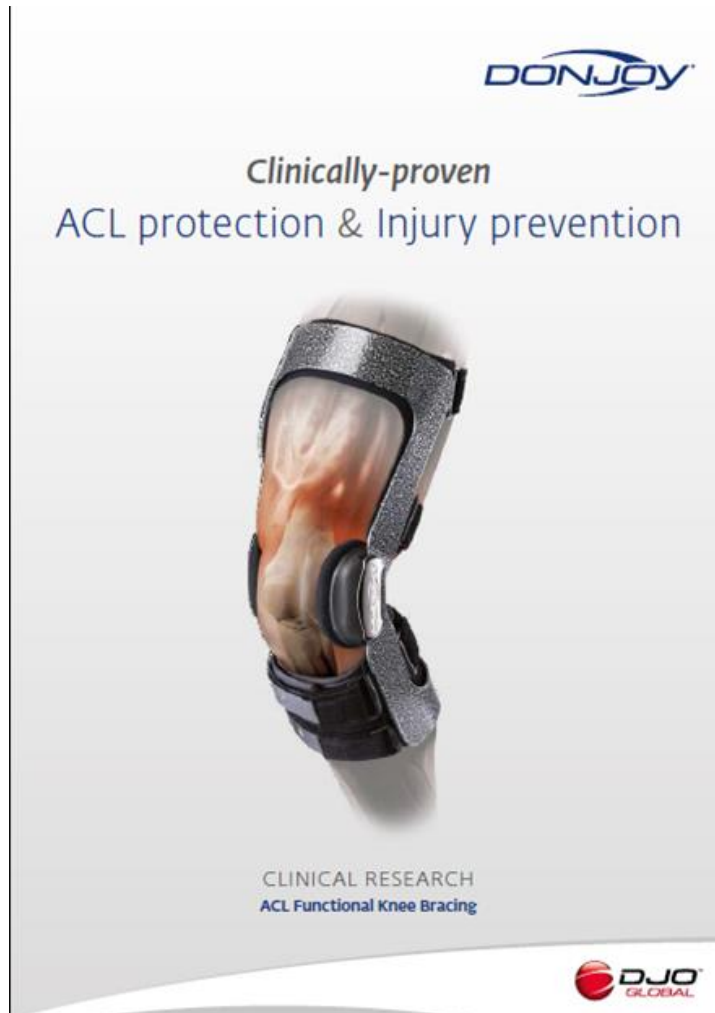- Use of semi-rigid knee braces in patients with knee OA contributes to general daily functioning, reduces pain, and increases the possibility to perform daily activities.
- For patients with OA, reduced pain enables increased mobility away from the home environment, resulting in a more active life and increased quality of life.
- Knee brace should be considered a useful non-surgical treatment method for use in patients with knee osteoarthritis.

### Key message

Based on patient feedback in this study, Donjoy OA Defiance can be considered a useful non-surgical treatment method for knee OA patients, offering pain relief, improved daily functioning and increased mobility and freedom of movement.

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